新冠抗原检测试剂产品手册
**Product Manual of
COVID-19 Antigen Detection Kit**

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COMPANY PROFILE

New Gene (Hangzhou) Bioengineering Co., Ltd. is located in Hangzhou, China. It is a high-tech company engaged in the research, development, manufacture and distribution of biological products. It is committed to creating biological materials such as antigens and antibodies, in vitro diagnostic reagents and related devices, and also the complete industrial chain of artificial intelligence assisted diagnosis system. The product line covers a full range of in vitro diagnostic products such as immune diagnosis, molecular diagnosis, and microbiological testing. NEWGENE has profound technical accumulation and unique technological advantages in the areas of early cancer screening, rapid detection of infectious diseases, and rapid screening of geriatric diseases.

NEWGENE's manufacturing system meets GMP standards for medical devices, and is certified with ISO13485 by British BSI. Relevant in vitro diagnostic reagent products have obtained the EU CE certification. NEWGENE is also a member on the "allow list" issued by Chinese Ministry of Commerce for anti-epidemic products exporting.

At present, NEWGENE COVID-19 Antigen Detection Kit has **registered in** many countries, including **Germany, France, Italy, Switzerland, Belgium, Portugal, Czech, Denmark, Hungary, Greece, Poland, Sweden, Moldova, Peru, Argentina, Ecuador, Kenya, Zimbabwe, South Africa, Malaysia, Thailand, Brunei, Congo etc.**, and passed the clinical verification of multiple national laboratories. The products are suitable for children under 14 years old.

Self-test Approval of EU

CERTIFICATE

EC Certificate No. 1434-IVDD-449/2021

**EC Design-examination
Directive 98/79/EC concerning
in vitro diagnostic medical devices**

Polish Centre for Testing and Certification certifies
that manufactured by:

**New Gene (Hangzhou) Bioengineering Co., Ltd.
Room 1606, Floor 16, Building 5, 688 Bin'an Road, Changhe Street,
Binjiang District, Hangzhou City, Zhejiang Province,
P. R. China**

in vitro diagnostic medical devices
for self-testing

COVID-19 Antigen Detection Kit - Nasal Swab

in terms of design documentation, comply with requirements
of Annex III (Section 6) to Directive 98/79/EC (as amended)
implemented into Polish law,
as evidenced by the audit conducted by the PCBC
Validity of the Certificate: from 11.08.2021 to 27.05.2024

The date of issue of the Certificate: 11.08.2021

The date of the first issue of the Certificate: 11.08.2021

Issued under the Contract No. MD-116
Application No: 239/2021
Certificate bears the qualified signature.
Warsaw, 11.08.2021
Module A1

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Manufacturer: New Gene (Hangzhou) Bioengineering Co., Ltd.
Address: Room 1606, Floor 16, Building 5, 688 Bin'an Road, Changhe Street,
Binjiang District, Hangzhou City, Zhejiang Province, P. R. China

EC Representative: SUNGO Europe B.V.
Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands
EC Certificate No.: 1434-IVDD-449/2021

Product Name: COVID-19 Antigen Detection Kit – Nasal Swab
Specification: 1Test/Box, 5Tests/Box, 25Tests/Box
Classification: Self Test (IVDD)
Conformity Assessment Procedure: Annex III (Section 6) to Directive 98/79/EC

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Signature: 

Name/ Position: Mingfu Li / General Manager

Date: 11/08/2021

Place: Hangzhou, Zhejiang, China

By Royal Charter

Certificate of Registration

QUALITY MANAGEMENT SYSTEM - ISO 13485:2016

This is to certify that: New Gene (Hangzhou)
Bioengineering Co., Ltd.
Room 1606, 16th Floor, No.5 Building
688 Bin'an Road
Binjiang District
Hangzhou
Zhejiang
310052
China

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Holds Certificate No: **MD 729179**

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 for the following scope:

Design and Development, Manufacture and Distribution of In-vitro Diagnostic Rapid Test Kit of Drug Abuse, Manufacture and Distribution of In-vitro Diagnostic Rapid Test Kit of Infectious Diseases.

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For and on behalf of BSI:

Gary E Slack, Senior Vice President - Medical Devices

Original Registration Date: 2020-07-27

Latest Revision Date: 2020-07-27

Effective Date: 2020-07-27

Expiry Date: 2023-07-26

Page: 1 of 1

...making excellence a habit.™

Self-Testing

CE 1434

COVID-19 Antigen Detection Kit - Nasal Swab

| No. | Components | 25 Tests/Box | 5 Tests/Box | 1 Test/Box |
|-----|--------------------------------------|--------------|-------------|------------|
| 1 | Test Card | 25 | 5 | 1 |
| 2 | Sample Extraction Tube & Tube Cap | 25 | 5 | 1 |
| 3 | Sampling Swab: <i>for Nasal Swab</i> | 25 | 5 | 1 |
| 4 | Package Insert | 1 | 5 | 1 |

25 Tests/Box

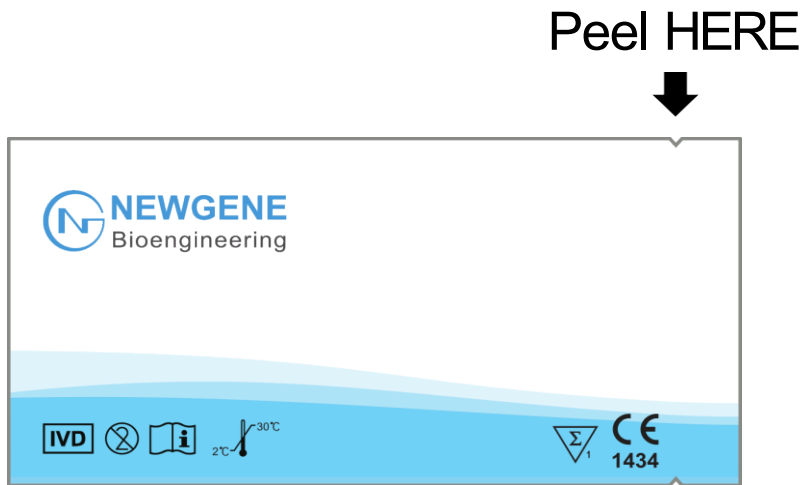
5 Tests/Box


1 Test/Box

COMPONENTS


Aluminum Foil Pouch

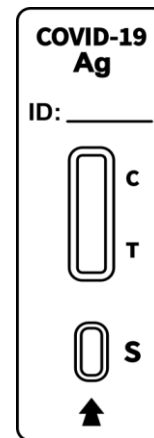
Test Card
(Packed in the pouch)

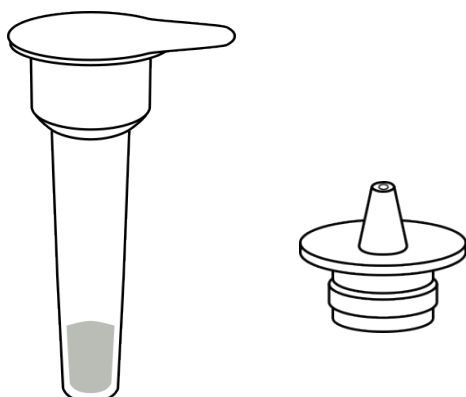
Desiccant
(Discard it.
Do not open)

**Sample Extraction Tube &
Tube Cap**

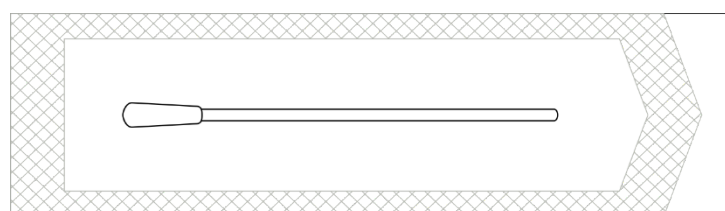Sampling Swab

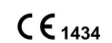Instructions for Use
COVID-19 Antigen Detection Kit

EN

For self-test use / Suitable for non-professionals to conduct self-test.

PRECAUTIONS BEFORE USING THE PRODUCT

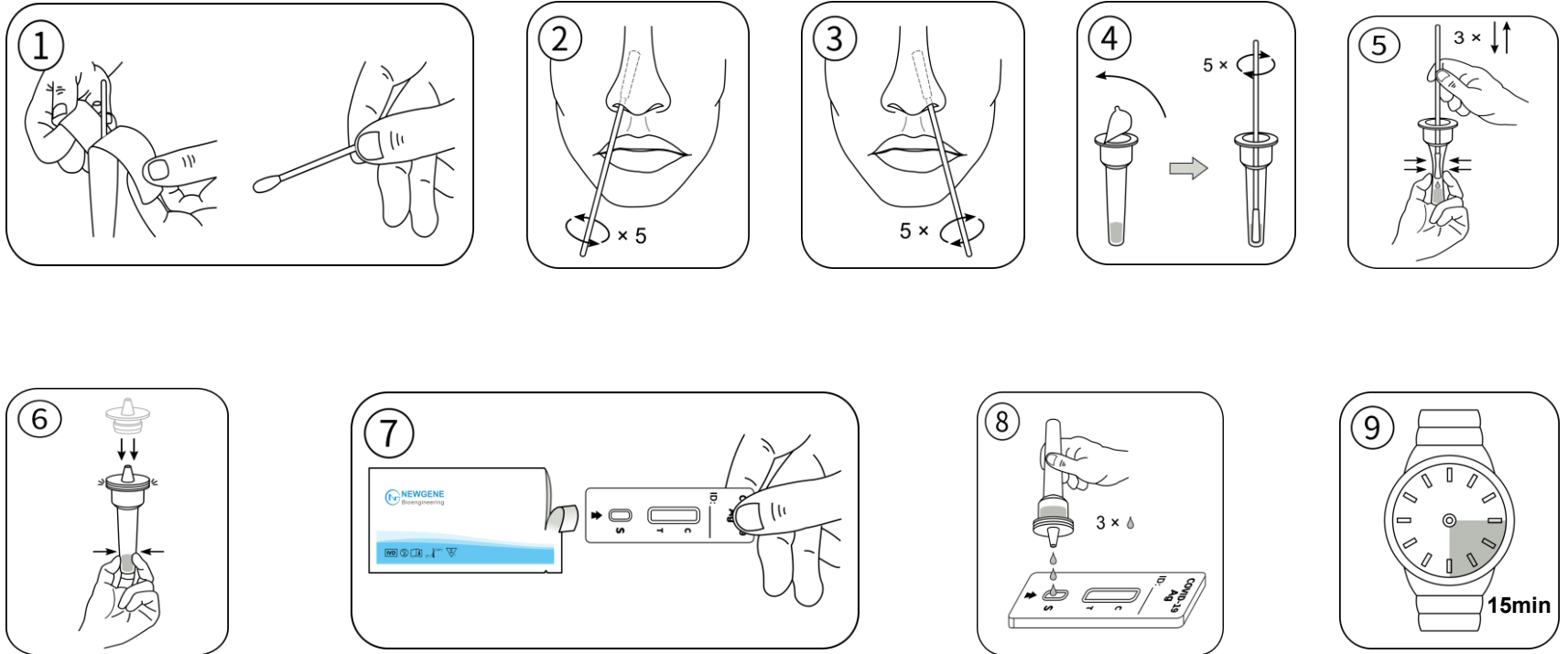
1. Read the instructions carefully prior to first use.
2. For people who are not able to perform the test themselves, the test should be conducted by the legal guardians.
3. For children under the age of 15, the self-test should be conducted under adult supervision.
4. This test detects SARS-CoV-2 antigen in nasal cavity secretions, which is collected by a sterile nasal swab.
5. For people who has recent nasal trauma or surgery, or has severe coagulopathy, gentle operation is required for nasal swab collection to avoid injuries to the nose.
6. Please use the components provided in the kit for testing. Do not use components from other sources.
7. Please use this product in a place with sufficient light, so as to interpret the results accurately.

PRECAUTIONS AFTER USING THE PRODUCT


1. If you get a positive result, please contact your family doctor, or seek help from a professional medical facility as soon as possible. You need a nucleic acid test to confirm viral infection.
2. A negative result cannot completely exclude the possibility of viral infection. Incorrect sampling or low viral load may also cause a false negative result.

Instructions for Use

TEST PROCEDURES

INTERPRETATION OF RESULTS

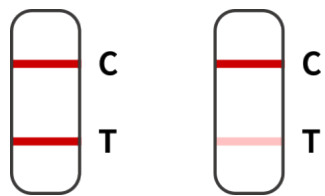
Positive (+): Red bands appear at both of T and C line in 15 to 30 minutes.

A white band at the T line should be considered as a negative result.

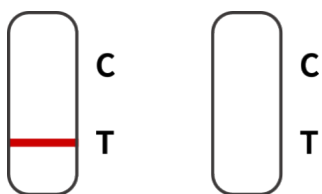Negative (-): A red band appears at C line while no red band appears at T line in 15 to 30 minutes after sample loading.

Invalid: If no red band appears at C line, it indicates that the test result is invalid.

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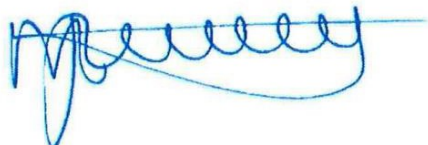
Dr. M.J. van de Velde

DECLARATION OF CONFORMITY

Regarding In Vitro Diagnostic Directive (98/79/EC)

Manufacturer: New Gene (Hangzhou) Bioengineering Co., Ltd.
Address: Room 1606, Floor 16, Building 5, 688 Bin'an Road, Changhe Street, Binjiang District, Hangzhou City, Zhejiang Province, P. R. China

EC Representative: SUNGO Europe B.V.
Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Product Name: COVID-19 Antigen Detection Kit
Product Code: COVID-19-NG08
Specification: 25Tests/Box 1Test/Box
Classification: Others (IVDD)

Conformity Assessment Procedure: Annex III of In Vitro Diagnostic Directive (98/79/EC)

We herewith declare that the above-mentioned products meet the requirements of In Vitro Diagnostic Directive (98/79/EC) and the following harmonized standards.

- | | |
|-------------------|---------------------|
| EN 23640-2015 | EN 13640:2002 |
| EN 980:2016 | EN 13641:2002 |
| EN ISO 14971:2019 | EN ISO 18113-1:2011 |
| EN 13612:2002 | EN ISO 18113-4:2011 |

On behalf of SUNGO Europe office, I confirmed we are EU REP of the company who issue this document.

Signature: 
Name/ Position: Mingfu Li / General Manager
Date: 29/09/2020
Place: Hangzhou, Zhejiang, China

 Authorized Signature (S)

COVID-19 Antigen Detection Kit (Nasal Swab)

For Professional Use

| No. | Components | 25 Tests/Box | 5 Tests/Box | 1 Test/Box |
|-----|-----------------------------------|--------------|-------------|------------|
| 1 | Test Card | 25 | 5 | 1 |
| 2 | Sample Extraction Tube & Tube Cap | 25 | 5 | 1 |
| 3 | Sampling Swab: for Nasal Swab | 25 | 5 | 1 |
| 4 | Package Insert | 1 | 5 | 1 |

25 Tests/Box

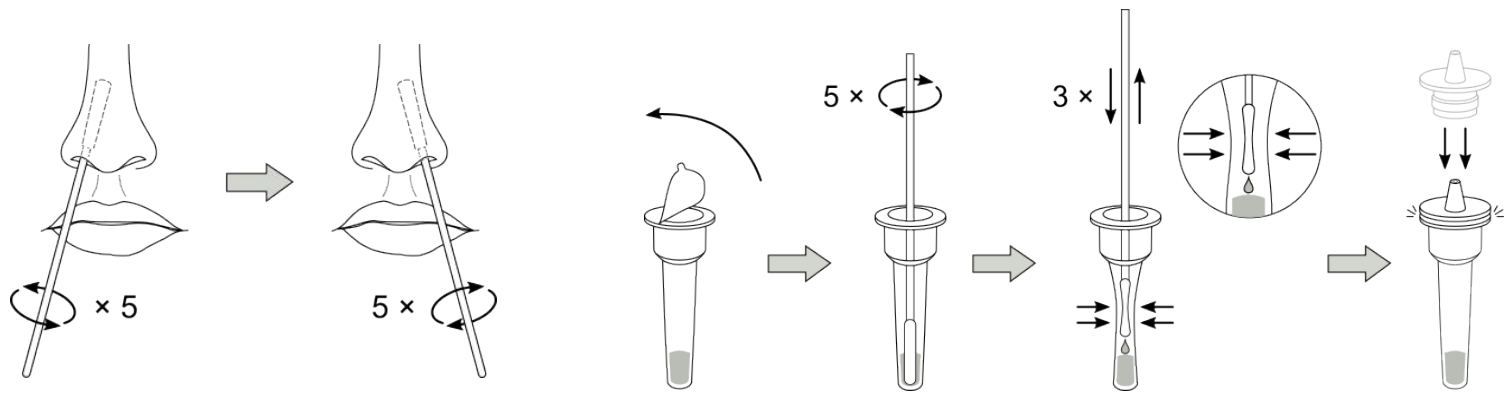
5 Tests/Box


1 Test/Box


Test Procedures: Sampling

Test Procedures: Detection

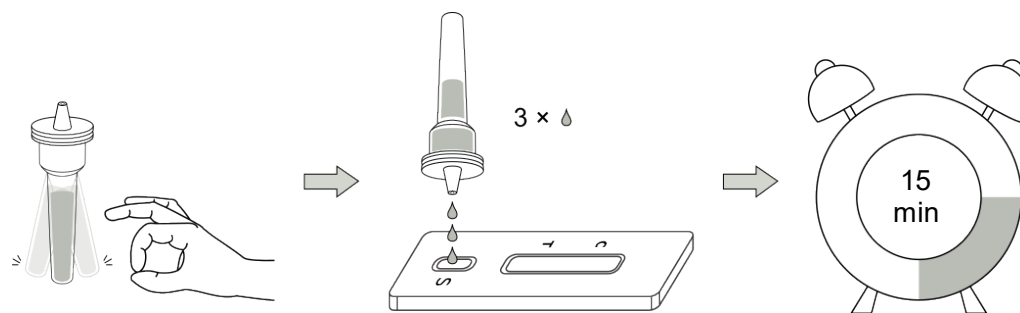Interpretation of Results

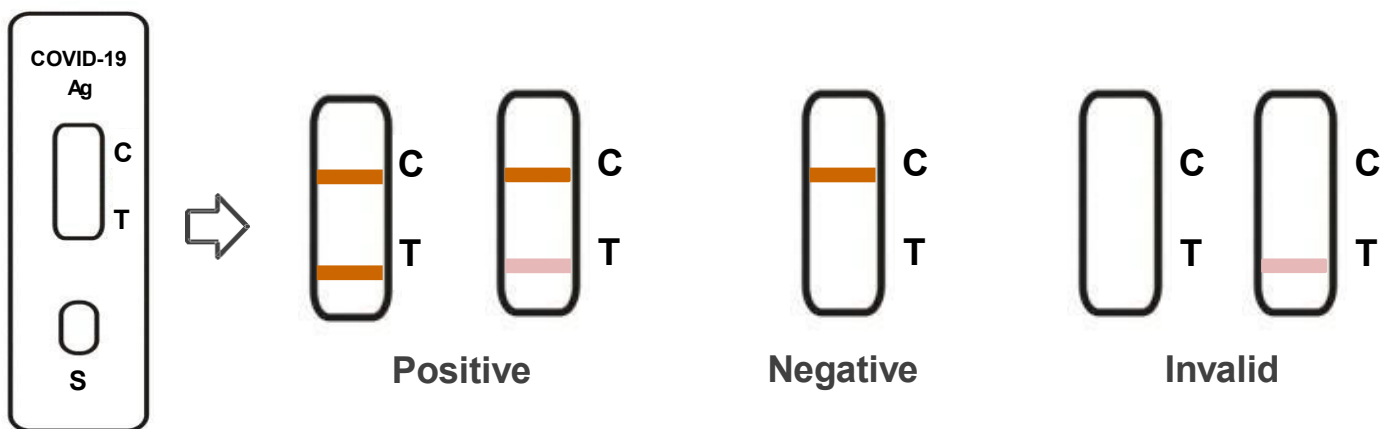Positive(+): Red bands appear at both of T and C line in 15 to 30 minutes. A white band at the T line should be considered as a negative result.

Negative(-): A red band appears at C line while no red band appears at T line in 15 to 30 minutes after sample loading.

Invalid: As long as no red band appears at C line, it indicates that the test result is invalid, and should retest the sample with another test card.

Performance

| Sensitivity | Specificity |
|-------------|-------------|
| 97.1% | 99.2% |

COVID-19 Antigen Detection Kit (Nasopharyngeal Swab)

For Professional Use

| No. | Components | 25 Tests/Box | 5 Tests/Box | 1 Test/Box |
|-----|---|--------------|-------------|------------|
| 1 | Test Card | 25 | 5 | 1 |
| 2 | Sample Extraction Tube & Tube Cap | 25 | 5 | 1 |
| 3 | Sampling Swab: <i>for Nasopharyngeal Swab</i> | 25 | 5 | 1 |
| 4 | Package Insert | 1 | 5 | 1 |

25 Tests/Box

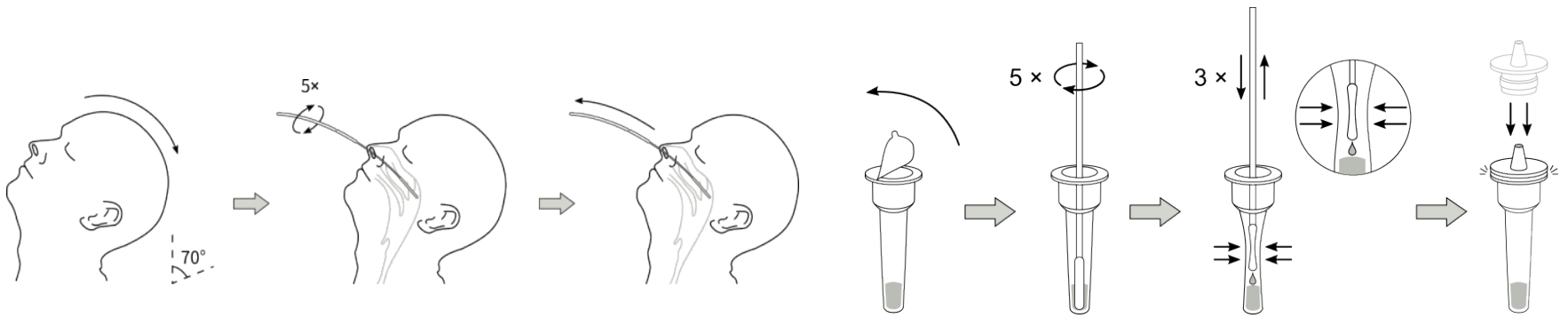
5 Tests/Box


1 Test/Box


Test Procedures: Sampling

Test Procedures: Detection

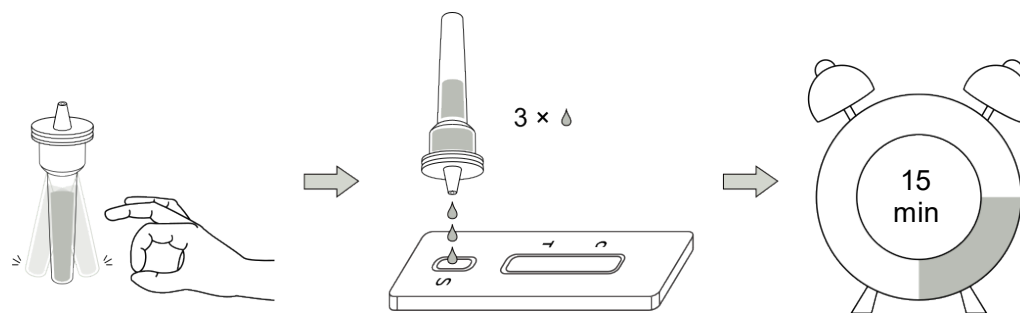Interpretation of Results

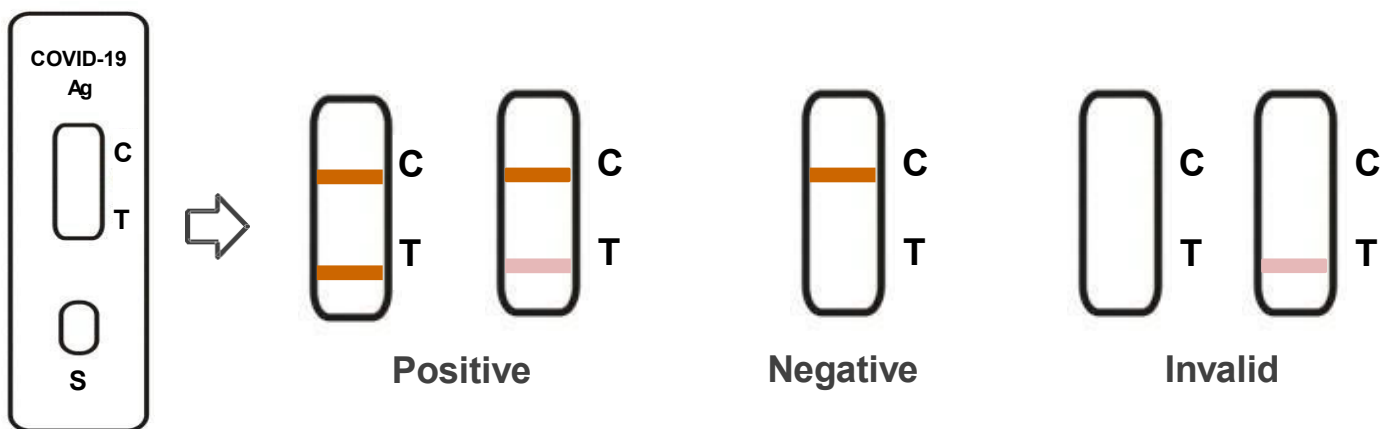Positive(+): Red bands appear at both of T and C line in 15 to 30 minutes. A white band at the T line should be considered as a negative result.

Negative(-): A red band appears at C line while no red band appears at T line in 15 to 30 minutes after sample loading.

Invalid: As long as no red band appears at C line, it indicates that the test result is invalid, and should retest the sample with another test card.

Performance

| Sensitivity | Specificity |
|-------------|-------------|
| 98.0% | 99.1% |

COVID-19 Antigen Detection Kit (Oropharyngeal Swab)

For Professional Use

| No. | Components | 25 Tests/Box | 5 Tests/Box | 1 Test/Box |
|-----|---------------------------------------|--------------|-------------|------------|
| 1 | Test Card | 25 | 5 | 1 |
| 2 | Sample Extraction Tube & Tube Cap | 25 | 5 | 1 |
| 3 | Sampling Swab: for Oropharyngeal Swab | 25 | 5 | 1 |
| 4 | Package Insert | 1 | 5 | 1 |

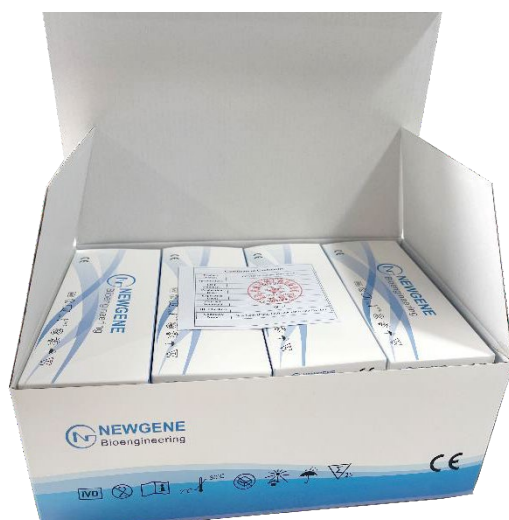
25 Tests/Box


5 Tests/Box


1 Test/Box


Test Procedures: Sampling

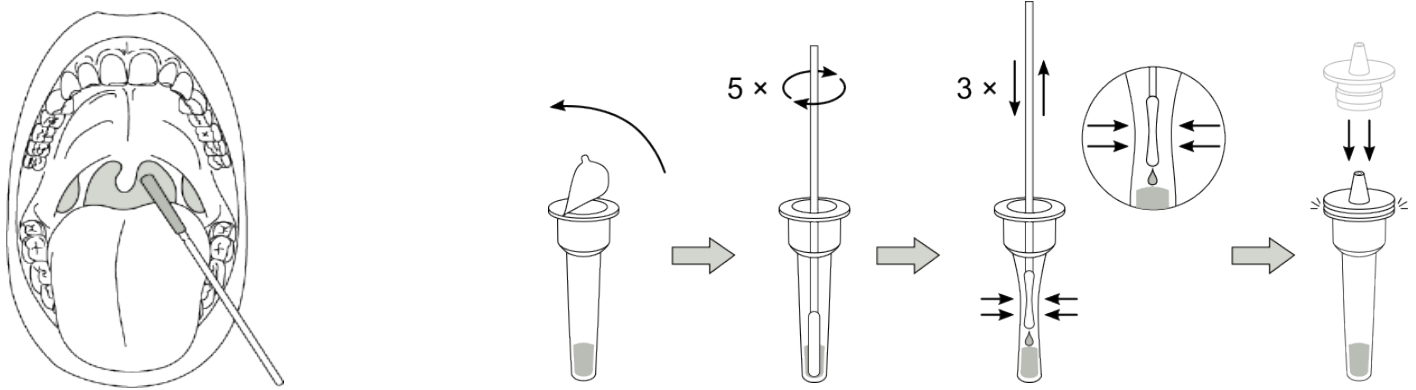Test Procedures: Detection

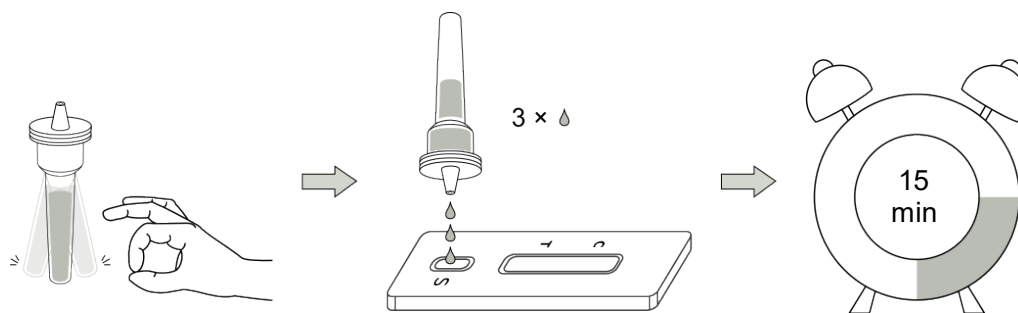Interpretation of Results

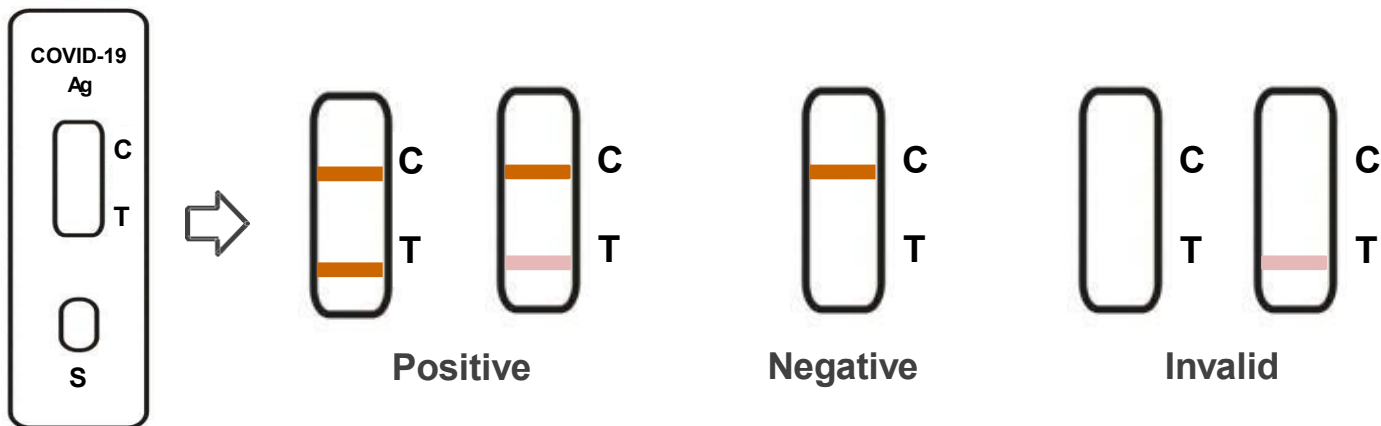Positive(+): Red bands appear at both of T and C line in 15 to 30 minutes. A white band at the T line should be considered as a negative result.

Negative(-): A red band appears at C line while no red band appears at T line in 15 to 30 minutes after sample loading.

Invalid: As long as no red band appears at C line, it indicates that the test result is invalid, and should retest the sample with another test card.

Performance

| Sensitivity | Specificity |
|-------------|-------------|
| 95.7% | 99.0% |

COVID-19 Antigen Detection Kit (Saliva)

For Professional Use

| No. | Components | 25 Tests/Box | 5 Tests/Box | 1 Test/Box |
|-----|-----------------------------------|--------------|-------------|------------|
| 1 | Test Card | 25 | 5 | 1 |
| 2 | Sample Extraction Tube & Tube Cap | 25 | 5 | 1 |
| 3 | Paper Cup | 25 | 5 | 1 |
| 4 | Sputum Dropper | 25 | 5 | 1 |
| 5 | Package Insert | 1 | 5 | 1 |

25 Tests/Box

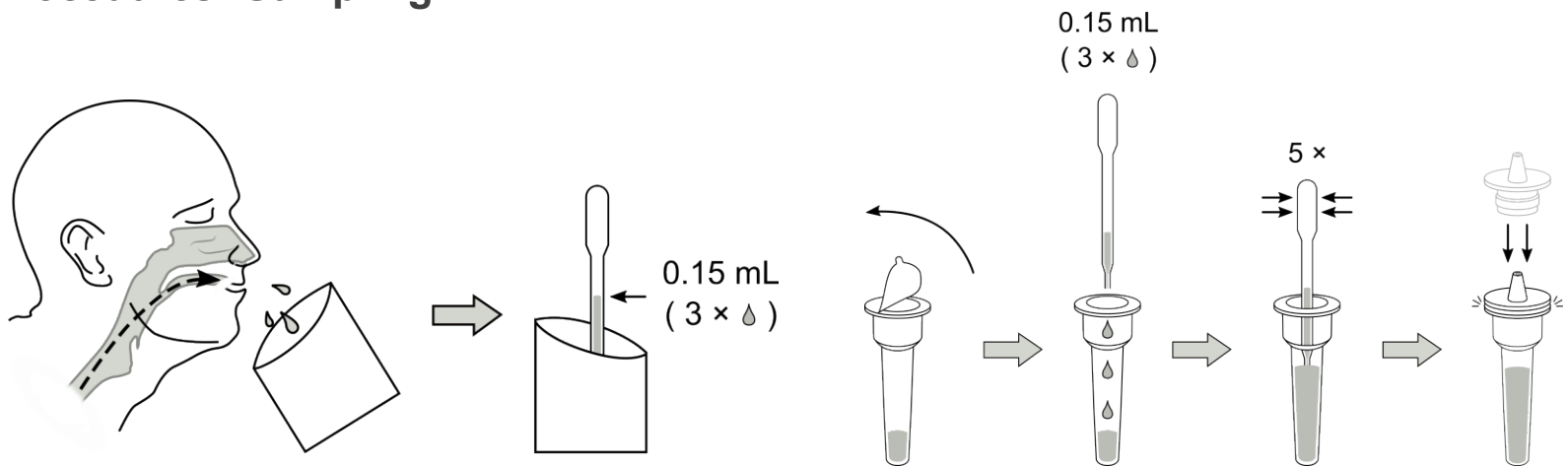
5 Tests/Box


1 Test/Box


Test Procedures: Sampling

Test Procedures: Detection

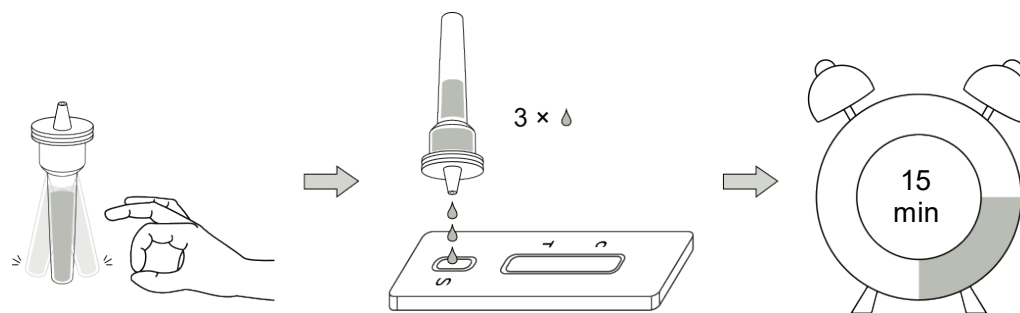Interpretation of Results

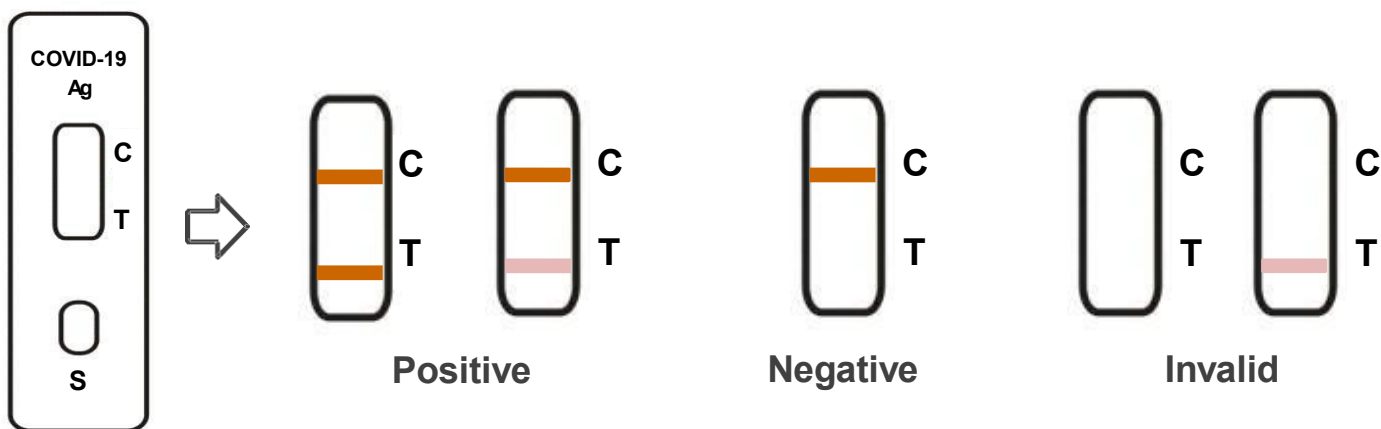Positive(+): Red bands appear at both of T and C line in 15 to 30 minutes. A white band at the T line should be considered as a negative result.

Negative(-): A red band appears at C line while no red band appears at T line in 15 to 30 minutes after sample loading.

Invalid: As long as no red band appears at C line, it indicates that the test result is invalid, and should retest the sample with another test card.

Performance

| Sensitivity | Specificity |
|-------------|-------------|
| 97.3% | 99.2% |

PACKAGING INFORMATION For Professional Use

Nasopharyngeal Swab: **NPS**

Nasal Swab: **NS**

Oropharyngeal Swab: **OS**

Saliva: **S**

25 Tests/Box

| Sample | NPS | NPS+S | NS | NS+S | OS | OS+S | S |
|--------------------|-------------|-------|----|------|----|------|-------------|
| Box (mm) | 230*140*80 | | | | | | 230*120*67 |
| Box weight (kg) | 0.36 | | | | | | |
| Carton (mm) | 585*485*425 | | | | | | 510*490*360 |
| Carton weight (kg) | 1.5 | | | | | | 1.3 |
| PCS/Box | 25 | | | | | | |
| Boxes/Carton | 40 | | | | | | |
| PCS/Carton | 1000 | | | | | | |
| Volume/Carton | 0.12CBM | | | | | | 0.09CBM |
| NW/Carton (kg) | 14.3 | | | | | | 15.6 |
| GW/Carton (kg) | 15.8 | | | | | | 16.9 |

5 Tests/Box

| Sample | S | NS | NS+S | NPS | OS | NPS+S | OS+S |
|-----------------------|-------------|----|------|-----|----|-------|------|
| Inner box (mm) | 193*85*42 | | | | | | |
| Outer box (mm) | 225*197*89 | | | | | | |
| Outer box weight (kg) | 0.5 | | | | | | |
| Carton (mm) | 470*410*470 | | | | | | |
| Carton weight (kg) | 1.4 | | | | | | |
| PCS/Inner Boxes | 5 | | | | | | |
| Inner Boxes/Outer Box | 20 | | | | | | |
| PCS/Carton | 500 | | | | | | |
| Volume/Carton | 0.09CBM | | | | | | |
| NW/Carton (kg) | 10 | | | | | | |
| GW/Carton (kg) | 11.4 | | | | | | |

1 Test/Box

| Sample | S | NS | NS+S | NPS | OS | NPS+S | OS+S |
|-----------------------|-------------|----|------|-------------|----|-------|------|
| Inner box (mm) | 143*83*15 | | | 170*66*15 | | | |
| Inner box weight (kg) | 0.027 | | | 0.028 | | | |
| Outer box (mm) | 305*197*88 | | | 277*182*112 | | | |
| Outer box weight (kg) | 0.8 | | | 0.83 | | | |
| Carton (mm) | 630*420*470 | | | 590*570*395 | | | |
| Carton weight (kg) | 1.8 | | | 2.2 | | | |
| PCS/Inner Boxes | | | | 25 | | | |
| Inner Boxes/Outer Box | | | | 20 | | | |
| PCS/Carton | | | | 500 | | | |
| Volume/Carton | 0.13CBM | | | 0.133CBM | | | |
| NW/Carton (kg) | 16.5 | | | 16.6 | | | |
| GW/Carton (kg) | 18.3 | | | 18.8 | | | |

PACKAGING INFORMATION Self-Testing

25 Tests / Box

| Sample | Nasal Swab |
|--------------------|-------------|
| Box (mm) | 230x120x75 |
| Box weight (kg) | 0.27 |
| Carton (mm) | 520X480X400 |
| Carton weight (kg) | 1.5 |
| PCS/Box | 25 |
| Boxes/Carton | 40 |
| PCS/Carton | 1000 |
| Volume/Carton | 0.1CBM |
| NW/Carton (kg) | 11 |
| GW/Carton (kg) | 12.5 |

5 Tests / Box

| Sample | Nasal Swab |
|-----------------------|-------------|
| Inner box (mm) | 122x68x49 |
| Outer box (mm) | 258X128X75 |
| Outer box weight (kg) | 0.38 |
| Carton (mm) | 550X530X395 |
| Carton weight (kg) | 1.4 |
| PCS/Inner Boxes | 5 |
| Inner Boxes/Outer Box | 40 |
| PCS/Carton | 1000 |
| Volume/Carton | 0.11CBM |
| NW/Carton (kg) | 15.6 |
| GW/Carton (kg) | 17kg |

1 Test / Box

| Sample | Nasal Swab |
|-----------------------|-------------|
| Inner box (mm) | 122X83X15 |
| Outer box (mm) | 255X170X105 |
| Outer box weight (kg) | 0.68 |
| Carton (mm) | 545X530X370 |
| Carton weight (kg) | 1.8 |
| PCS/Inner Boxes | 25 |
| Inner Boxes/Outer Box | 20 |
| PCS/Carton | 500 |
| Volume/Carton | 0.10CBM |
| NW/Carton (kg) | 12.45 |
| GW/Carton (kg) | 14.25 |

EUROPEAN COMMISSION
DIRECTORATE-GENERAL FOR HEALTH AND FOOD SAFETY

Public health, country knowledge, crisis management
Health Security

EU health preparedness:

A common list of COVID-19 rapid antigen tests and a common standardised set of data to be included in COVID-19 test result certificates

Agreed by the Health Security Committee

This document was agreed by the HSC on 17 February 2021

Annex I

Common list of COVID-19 rapid antigen tests

A first update was agreed by the HSC on 10 May 2021; A second update was agreed by the HSC on 16 June 2021; A third update was agreed by the HSC on 7 July 2021; A fourth update was agreed by the HSC on 14 July 2021; A fifth update was agreed by the HSC on 23 July 2021.

IMPORTANT: A (interim) grace period of 8 weeks applies whenever updates are made to Annex I, the common list of COVID-19 rapid antigen tests

| | | | | | | | | | | | |
|--|--------------------------------|-----|-------------------------------|---|--|-------------------|--|-------------------|--|------|--------------|
| New Gene (Hangzhou) Bioengineering Co., Ltd. | COVID-19 Antigen Detection Kit | Yes | 98% sensitivity Nasal swab | DE: Positive evaluation by Paul-Ehrlich-Institut (sensitivity of 92,5% at <Ct30 and 100% at <Ct25) | | DE ^[2] | | DE ^[2] | | 1501 | 16 June 2021 |
|--|--------------------------------|-----|-------------------------------|---|--|-------------------|--|-------------------|--|------|--------------|

Validation protocol for the Newgene bioengineering COVID-19 Rapid Antigen Test

Testing laboratory:
Molecular Diagnostic Laboratory
Via Petrini 2
CH-6900 Lugano
Switzerland

Discussion and Conclusion.

In conclusion, the Newgene antigen detection test is highly precise and accurate (**100% specificity and 95.1% sensitivity**), it is non-invasive and mimics the PCR results very closely where PCR is considered the golden standard technique. The device can easily be operated and used by non-medically trained personnel, does not need a laboratory setting and could be intended for regular use by regular people.

Lugano, June 18th 2021.
Dr. G. Soldati
CEO
Molecular Diagnostic Laboratory
Via Petrini 2
CH-6900 Lugano, Switzerland

Evaluation Report (Children under the age of 14)

Discussion

In the present study, the COVID-19 Antigen Detection Kit - Nasal Swab by New Gene (Hangzhou) Bioengineering Co., Ltd. has shown highly reliable performance in sample from children under the age of 14. Compared to conventional RT-PCR tests, the rapid antigen tests can meet a wider range of test needs. As the antigen tests take only 20 to 30 minutes, they are feasible for use in emergency scenarios where a test result is demanded immediately. Also, the rapid antigen tests doesn't require special instruments and training to use, they are also capable for resource limited scenarios like point-of-care testing and self-test by laypeople. Therefore, the implementation of rapid antigen test may totally change the strategies to control COVID-19. Community residents can conduct the rapid antigen test in a frequent manner, like twice or three times a week, to identify COVID-19 cases in the early stage of infection. This strategy may help to stop the transmission of COVID-19 as early as possible. In summary, the COVID-19 Antigen Detection Kit - Nasal Swab has shown satisfying sensitivity, specificity, and total accuracy in the present evaluation. It can be used as a rapid tool to assist the early diagnosis of COVID-19 cases in children under the age of 14.

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Evaluation Report (Saliva)

Result Analysis:

The test results of RT-PCR and the COVID-19 Antigen Detection Kit are summarized in 2x2 table below.

| | | RT-PCR Reagent | | Total |
|--------------|----------|----------------|----------|-------|
| | | Positive | Negative | |
| Antigen Test | Positive | 204 | 0 | 204 |
| | Negative | 16 | 200 | 216 |
| Total | | 220 | 200 | 420 |

Sensitivity (%) = $204 \div (204 + 16) \times 100\% = 92,73\%$; 95% CI: 88,46% ~ 95,79%

Specificity (%) = $200 \div (0 + 200) \times 100\% = 100,0\%$; 95% CI: 98,17% ~ 100,00%

Total accuracy (%) = $(204 + 200) \div (204 + 0 + 16 + 200) \times 100\% = 96,19\%$;

95% CI: 93,89% ~ 97,81%

The 95% Confidence Intervals of sensitivity, specificity, and total accuracy are calculated following the binomial distribution.

Analysis results by Kappa consistency test shows that the Kappa value = 0,9239 (95% CI: 0,8875 ~ 0,9604). As $Kappa \geq 0.75$, it suggests good consistency between the COVID-19 Antigen Detection Kit and the COVID-19 RT-PCR detection reagent. Therefore, the COVID-19 Antigen Detection Kit has shown reliable performance on detecting SARS-CoV-2 virus in saliva samples from suspected COVID-19 patients.

(+++) - positive - intensity of line T similar or higher than line C

(++) - positive - intensity of line T lower than line C

(+) - positive - border intensity of line T

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Registration or Allowed List (Partially)

Self-test Approval in Malaysia

Home / Announcement / SELF-TEST COVID-19 TEST KIT FOR CONDITIONAL APPROVAL (APPROVED)

SELF-TEST COVID-19 TEST KIT FOR CONDITIONAL APPROVAL (APPROVED)

SELF-TEST COVID-19 TEST KIT FOR CONDITIONAL APPROVAL (APPROVED)

The list of Self-Test Covid-19 Test Kit that is approved for Conditional Approval based on the decision on the consensus of the Covid-19 Test Kit Expert committee is as follows.

All test submissions are scored according to: -

- the manufacturer reported clinical and analytical performance evidence.
- the evaluation test results from testing facilities are according to the committee evaluation criteria set by Clinical expert panels.
- Supporting Documents for COVID-19 IVD Test Kits Conditional Approval.

The use of COVID-19 self- test kit shall be limited for screening purpose only and all test results need further confirmation using RT-PCR.

This test kit can be supplied by registered pharmacists or private healthcare facilities.

Below is the list of all tests that have been selected to date and the status is Conditional Approval (please note: list is updated on a routine basis).

Notes:

*Sample type is based on testing facility evaluation report.

| NO | COMPANY NAME | PRODUCT NAME | MANUFACTURER | IDENTIFIER | DETECTION | SAMPLE TYPE |
|----|------------------------|---|---|---------------|-------------------------|----------------------|
| 11 | Dewina Consult Sdn Bhd | NEWGENE Bioengineering COVID-19 Antigen Detection Kit | New Gene (Hangzhou) Bioengineering Co., Ltd. P.R. China | COVID-19-NG08 | RTK-Antigen (Self-test) | Saliva or Nasal swab |

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Kenya

MINISTRY OF HEALTH
PHARMACY AND POISONS BOARD

(Section 3B(2)(e) of the Pharmacy and Poisons Act, Cap 244 Laws of Kenya)

IN-VITRO DIAGNOSTIC EMERGENCY USE AUTHORIZATION

This Emergency Use Authorization is issued to **New Gene (Hangzhou) Bioengineering Co., Ltd.**, for distribution and sale of **Novel Coronavirus Antigen Detection Kit (Colloidal Gold)**

| | |
|---------------------------------------|--|
| Emergency use Authorization (EUA) No. | MD/2021/7674 |
| EUA valid until | End of COVID-19 Pandemic or EUA revocation |

Kenya

2020-10-30 14:25:20

SAHPRA Head Office
Building A
Lofos Park
2nd Floor
Kirkness Road
Atteridgeville
0083

Examiner: Medical Device Unit
Ref: N/A
Email: shah@sahealthproducts.gov.za
Tel No: 0027 71020
License: 000020040_v2

SAHPRA Head Office
Building A
Lofos Park
2nd Floor
Kirkness Road
Atteridgeville
0083

Dear Sir/ Madam,

RE: AUTHORISATION FOR THE SALE OF COVID-19 Antigen POINT-OF-CARE TEST KIT – SARS-CoV-2 Antigen Rapid

**COVID-19 RAPID TESTS KITS (ART)
AUTHORISED FOR USE IN BRUNEI DARUSSALAM.**

The listed Covid-19 antigen rapid test kits that are recommended and authorized for use are based on the evaluation done by Ministry of Health, Brunei Darussalam. The results of the evaluations are determined according to the clinical and analytical performance of the test kits (sensitivity and specificity claimed by the manufacturers), safety standards, quality and efficacy of the test kits.

Ministry of Health, through the Department of Laboratory Services will continue to update the list of authorized Covid-19 rapid test kits in order to ensure the supplied antigen rapid tests kits are meeting the required standards.

This list is updated as at 6 September 2021.

| NO | PRODUCT NAME | MANUFACTURER | DETECTION | SAMPLE TYPE |
|----|---|--|-----------|---------------------------|
| 8 | NEWGENE Bioengineering COVID-19 Antigen Detection Kit | New Gene (Hangzhou) Bioengineering Co., Ltd. China | Antigen | Sputum/ Nasopharyngeal |

South Africa

Brunei